

Congratulations!!

On your New DeBug Beach Walker

Please read the following few pages for Assembly and Maintenance



Rollator Walker
with Wheeliez
All-Terrain
Wheels

Instruction Manual And Safety Instructions

Thank for your purchase of the DeBug Beach Walker
We build the DeBug Beach Walker with the highest quality materials,
keeping in mind the comfort and positioning of the rider.
If any repairs should be required, please do not attempt them without
proper technical training. Please contact your nearest distributor for any
repairs, or adjustments needed

This Instruction Manual contains the following

Index		Page
	1 Tools Needed	2
	2 Assembly/ Disassembly of Walker	3
	4 Seating Position	4
	5 General Maintenance Instructions	5

All information in this manual is subject to change without prior notice.
Please contact your distributor for the latest version.

Thank-you
The De-Bug Mobility Products Team

Tools Needed



To start using your Multi Terrain Walker

1. Unbox The Walker and remove all packaging paper
2. Remove the Zip ties that suspend the rear wheels during shipping
3. Open the hardware bag and remove the nylon insert nuts from the rear wheel bolts
4. Insert the rear wheel assembly into the rear Walker tubes as shown in photo above
- 5 .Insert the rear wheel bolts, secure with their nylon insert nuts and hand tighten nuts
6. Using a flat Edge like 3 foot yardstick align the wheels so they are parallel. Tighten the nuts to secure the rear wheels
7. Insert the right and left handles, adjust the desired height and secure with knobs included Hardware bag. Generally, the optimal put handle height is just above the hip of the user
8. The brake handles squeeze to break, and push down to lock

The seat auction should only be used with the front Caster in the orientation shown, and with the brakes in the lock position



Walker in Seated Position
Note the orientation of
Casters are inverted
for stability. Do not sit
on walker with the casters
in any position other than
shown in this Photograph.

General Maintenance

It's so easy to maintain your DeBug beach wheelchair! Just follow these simple instructions, and you will be able to enjoy your chair year after year.

1. Do not submerge the walker in water of any kind!!
2. Check tire pressure before each use. These wheels are meant to be pliable and somewhat "Soft" to the touch. Recommended tire pressure is 2.5 to 4 PSI. These specifications are molded on each wheel.
3. Clean the walker of any loose debris, sand or dirt after each use
4. If at anytime you need to lubricate any part of the chair, DO NOT USE WD-40, as it will attract dirt and sand causing possible damage to the wheel bearing/bushings. We recommend P.T.F.E Dry Lube, on all wearing components. This dry film is a superior lubricant and repels debris and water . Marine and Performance parts stores likely stock this locally.